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1. Allgemein:

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Operating Instructions BA

1. General:

1.1. **Product name:** **Preformed bite blocks to determine the patient's bite situation in conjunction with setting and tunneling wax for better fixation and rearrangement of the teeth.**
Indication

1.2. Scope of delivery:			Item no.	Yeti
	BITE RIMS aroma free yellow soft	100 pc.	82300	
	BITE RIMS aroma free green medium hard	100 pc.	82850	
	BITE RIMS aroma free orange medium hard	100 pc.	82900	
	BITE RIMS aroma free red hard	100 pc.	82800	
	BITE RIMS aroma free pink extra hard	100 pc.	82350	614-0100
	BITE RIMS flavored lemon yellow soft	96 pc.	62100	610-0100
	BITE RIMS flavored apple green medium hard	96 pc.	62200	612-0100
	BITE RIMS flavored orange medium hard	96 pc.	62300	613-0100
	BITE RIMS flavored strawberry red medium hard	96 pc.	62400	611-0100
	BITE RIMS flavored vanilla dentin	96 pc.	62500	
	BITE RIMS flavored vanilla dentin XXL	50 pc.	62600	
	Tribos tunnel wax white strong sticky	100 gram	78200	
	Setting wax black slightly sticky 3mm	4 plates	85200	

1.3. **Manufacturer address:**  GEBDI Dental-Products GmbH, Industriestrasse 3a, 78234 Engen, Germany
 Phone: 07733-941016 FAX: 07733-6434 info@gebdi-dental.com

1.4. **Storage conditions:** Storage at room temperature min. +10° C to + 30° C
 Protect from direct sunlight
 At temperatures below 8° C especially at temperatures in the minus range, increased susceptibility to breakage.
 Protect packages from impact and fall

1.5. **Contraindication:** In individual cases in case of incompatibilities/allergies to all components of the wax raw materials mentioned in the safety data sheet. For example kerosenes, beeswax, carnauba wax.

2. Intended use:

Bite blocks for use in the dental laboratory, especially in complete and partial denture prosthetics. For taking a bite or determining the bite situation, short-term in the patient's mouth. The shape of the bite blocks can be changed by heating (open Bunsen flame/water basin). The softening or liquefaction of the wax for the actual bite situation in the patient's mouth must **always** be carried out outside the patient's mouth with suitable instruments by trained and suitable specialist personnel. Only insert sufficiently large and not too hot pieces into the patient's mouth to avoid swallowing, choking or burning the mucous membranes.

Setting wax or tunnel wax (Tribos 501 primary morphology/tunnel shape particularly suitable) can be used for faster and easier fixation or changeover of the confection teeth.

In case of improper use, all warranty and liability claims become void!

3. Risks and dangers in the application:

1. Ensure that persons not familiar with the product (e.g. children) do not come into contact with it.
2. Avoid hot wax drops on skin surfaces. Avoid inhalation of wax fumes during burning/burning out. (Protective clothing as required)
3. Wax may only be placed in the patient's mouth for a short time and not permanently.
4. Connect the bite block, setting wax or tunnel wax to the base body in such a way that there is no risk of the patient swallowing or choking. Ensure that the setting wax and tunnel wax are sufficiently connected to the ready-made teeth.

4. Instruction manual:

Document:	Created on / by:	modified on / by:	Revision:	Released on / by:	Page number:
BA	19.11.1997 / TB	04.04.2022 / TB	1	06.04.2022 / TB	Page 1 from 2

Operating Instructions BA

4.1. Preparation

Open the packaging on the upper side at the corresponding marking/lid.

4.2. Application

Cut bite blocks to length with a heated wax knife or suitable instrument and align and adjust on the working model after slight heating (Bunsen burner/water bath). Model onto the base body and ensure sufficient adhesion by means of liquefied wax. Smallest pieces of bite blocks should be avoided due to low bonding to the base body.

Cut setting wax and tunnel wax to desired size, apply confection teeth to securely fuse against breaking out. Tunnel and setting wax also fuse well towards the wax base.

4.3. After application

Return reusable bitewall residues, set wax and tunnel wax residues to the container, close lid, protect from dust.

5. Disposal:

Can be disposed of in residual waste in accordance with the applicable regulations. The outer packaging can be recycled (Dual System).

6. Shelf life:

Consumables should generally be used up quickly and immediately within a few months after opening the original packaging. The flavors are food safe and volatilize!

7. Symbols

	Manufacturer
CH-REP	Authorized person Switzerland
	CE Mark
R _x only	FOR DENTAL PROFESSIONALS ONLY (USA)
	Non-sterile product
	Order number
	Batch number
	Date of manufacture
	Usable until

Document:	Created on / by:	modified on / by:	Revision:	Released on / by:	Page number:
BA	19.11.1997 / TB	04.04.2022 / TB	1	06.04.2022 / TB	Page 2 from 2

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